


# QuickTEST<sub>DEA 1</sub>


1

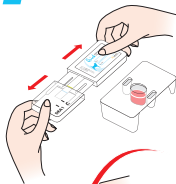


5



2

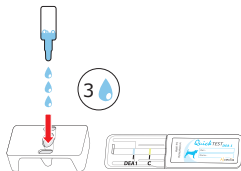


6

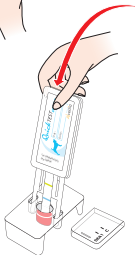

3

3

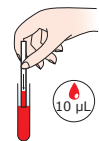


Packed Red Blood cells  
Or whole blood

- EDTA
- ACD
- CPD

Do not use Heparin


7

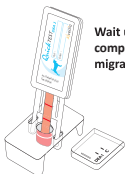

Wait until  
complete  
migration

2 to 4 min

4

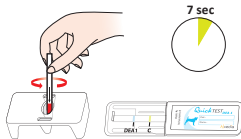

7 sec


8




Read immediately after testing



C = control

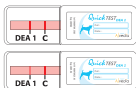

Weak DEA 1 line

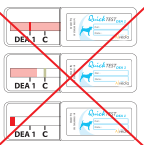

Contact us to ask for the  
«Troubleshooting procedure»

For more information, Please contact Asia Vet Medical Limited.

Asia Vet Medical Limited  
Tel: (852) 2335 9200  
Fax: (852) 2335 0008  
Email: sales@avmhk.com



Alvedia  
Your partner in health